## Wrist brace (splint) for either hand

Prusa Research

[VIEW IN BROWSER](#)

updated 7. 4. 2022 | published 29. 3. 2022

### Summary

A more rigid brace, suitable for broken wrist treatment, i.e. fracture of either the adjacent radial bone, ulnar bone...

3.99 hrs

1 pcs

0.30 mm

0.40 mm

PLA

76 g

Prusa  
MK3/S/S+

[Healthcare](#) > [Medical Tools](#)

Tags: [brace](#) [emergency](#) [ukraine](#) [surgery](#) [splint](#) [fracture](#)

### Wrist brace (splint) for either hand

A more rigid brace, suitable for broken wrist treatment, i.e. fracture of either the adjacent radial bone, ulnar bone, or wrist bones (excluding the scaphoid bone fracture, which requires fixation of the thumb as well) and relief from pain caused by a sprained wrist.

Use undercast padding, fixated with velcro straps (there are cutouts for inserting them), or wrap the brace in a self-adhering (or regular) bandage.

**Note:** Future iterations will have rounded edges, to make the brace more comfortable for prolonged use.

**Disclaimer:** This product is not a medical device. It has been created in answer to rapidly increased demand and has not been approved by any

relevant authority. The product shall not substitute professional medical treatment and its improper use can cause injury. Always consult your physician for personalized medical advice before its use.

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PLA 0.40 mm 0.30 mm 3.99 hrs 76 g Prusa MK3/S/S+

## License ©

This work is licensed under a  
[Creative Commons \(4.0 International License\)](#)

**Attribution-NonCommercial**

- 
- ✗ | Sharing without ATTRIBUTION
  - ✓ | Remix Culture allowed
  - ✗ | Commercial Use
  - ✗ | Free Cultural Works
  - ✗ | Meets Open Definition